

EN (English)

VITREO RETINAL CANNULA

DEVICE DESCRIPTION

This is a disposable ophthalmic cannula of different gauge and needle lengths. This device is supplied sterile in peel open pouches.

INTENDED USE / INDICATION FOR USE

This product is used for specific procedure in vitreoretinal surgery. Many of these innovative products have distinct feature including micro tip extensions and ultra-thin wall cannulas.

CONTRAINDICATIONS

The device is contraindicated for patients with known silicon and plastic sensitivity.

HOW SUPPLIED

Vitreo Retinal Cannula is supplied sterile, packed in a peel open pouch. Either Fifty (50) or Ten (10) such pouches are enclosed within an outer printed box. The overall packing contains Vitreo Retinal Cannula and medical literature insert, in an outer enclosing box, with external identification stickers.

CAUTIONS

The device should not be used in patients who show hypersensitivity to any of the components of the preparation.

⚠ WARNING

- Madhu Instruments Pvt. Ltd does not hold any responsibility if the device is reused. Reusing the device may cause serious problems to patients are as under:-
 - a. The used Vitreo Retinal Cannula tip might be blunt or has burrs. Do not use the cannula if the cannula tip is blunt or has burrs.
- Only qualified medical practitioner should use the device.
- Do not use the device if the sterilized package is open or damaged.

EXPIRATION DATE


The expiration date of the product is the sterility expiration date. The product should not be use after the indicated sterility expiration date.

RETURN OF DAMAGED PRODUCT

Return the product in its original packing identified by the batch number, purchase information, your reference and reason for return. Please contact your local distributor office regarding product return/exchange.

EXPLICATION OF SYMBOL:

Symbol	Meaning
	Caution
	Manufacturer
	Date of Manufacture
	Do not re-sterilize
	Consult Instructions for use
	Keep away from sunlight
	Don't re-use
	This side up
	Do not use hook

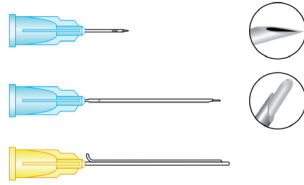

- Sterility is guaranteed only when the packing is neither opened nor damaged.
- Do not re-sterilize the device by any method. Madhu Instruments assumes no liability for the devices that have been re-sterilized.
- Do not reuse the device.
- Read instruction before use.
- Store at specified temperature and humidity.
- The used Vitreo Retinal Cannula and pouch should be disposed off in compliance with the local regulatory requirements.
- Do not use the device after sterility expiry date.
- For External use only
- Keep out of reach of children.

PRECAUTIONS

- When handling the product care should be taken to avoid damage from handling. Avoid crushing and crimping damage to the product due to application of surgical instruments such as forceps etc.

DIRECTION FOR USE

- Open the sterile Vitreo Retinal Cannula pouch (BY AN UNSTERILE PERSON) and drop the inner pouch on a sterile surface without touching it.
- Use the Vitreo Retinal Cannula as per the intended use and correct surgical procedure.

REPORTING

Adverse events and/or potentially sight-threatening complications that may reasonably be regarded as product related and that were not previously expected in nature, severity or incidence must be report to Madhu Instruments Pvt. Ltd.

DISPOSAL OF THE USED DEVICE

The used device should be disposed off in compliance with the local regulatory requirements.

Symbol	Meaning
	Batch Code
	Use by date
	Temperature Limit
	Do not use if package is damaged
	Humidity Limitation
	Sterilised using Ethylene Oxide
	Handle with care
	Maximum stacking
	Medical Device
	Catalogue Number

MADHU INSTRUMENTS PVT. LTD.

Mfg. Facility: A 260, Okhla Industrial Area Phase-I, New Delhi-110020 India.
 Regd. Add.: F-90/3D, Okhla Industrial Area Phase-I, New Delhi-110020 India
 Customer Care No.: +91-11-42701030, 42701031
 Email: customercare@madhuinstruments.com
 Website: www.madhuinstruments.com