

EN (English)

i-STAND PLUS

Eyeball Stand with Fixation Head

DEVICE DESCRIPTION

An Eyeball Stand with Fixation Head consists of a stainless-steel Eye Holder fixed inside a plastic Fixation Head to give a more natural feel of the human face to the user. The Eye Holder has a cavity for keeping the eyeball in place whereas one of the two Eye Rings of diameter 19.00 mm and 21.00 mm is used to fix the eyeball firmly and regulate the intraocular pressure. Eye Ring is chosen depending upon the size of the eyeball being used.

INTENDED USE/ INDICATION FOR USE

Eyeball Stand with Fixation Head is used for securely fixing human / animal eye during micro-surgical training.

MATERIALS USED

Stainless Steel, HIPS Plastic

INTENDED USER

Any of the registered medical doctor/medical professionals.

STORAGE CONDITIONS

Store at Room Temperature

HOW SUPPLIED

Eyeball Stand with Fixation Head are supplied non-sterile in corrugated box. The overall packing contains Eyeball Stand with Fixation Head and medical literature inserted in an outer enclosing box, with external identification stickers.

CAUTION/WARNING

- The use of this product is restricted to a qualified doctor or medical practitioners, medical professionals
- The product is use as a training device only.

- Read instructions before use.
- For External use only.
- Keep out of reach of children.



PRECAUTIONS

- Ensure before use that product/packaging is not tampered or damaged.
- When handling the product care should be taken to avoid damage from handling.

DIRECTION FOR USE

- Loosen the Side Screw Knob, pull out the Eye Ring and keep it aside.
- Place the eyeball into the Eye Holder with cornea facing upwards.
- Select the Eye Ring of appropriate size according to the eyeball and insert the Eye Ring Pins into the holes adjacent to the Eye Holder.
- After centering the cornea, further press the Eye Ring to get the desired Intraocular Pressure (IOP) and lock using the Side Screw Knob.
- Once the eyeball is properly fixed and desired IOP is achieved, proceed with intended surgical/ training procedure.

RETURN OF DAMAGED PRODUCT

Return the product in its original packing identified by the batch number, purchase information, your reference and reason for return. Please contact your local distributor office regarding product return/exchange.

DISPOSAL OF THE USED DEVICE

The used device should be disposed off in compliance with applicable local, state and country's laws and regulations.

REUSE OF DEVICE

The device should undergo a detailed multi step process of cleaning & sterilization before reuse

EXPLICATION OF SYMBOLS:

Symbol	Meaning
	Caution
	Manufacturer
	Date and Country of Manufacture
	Consult instructions for use
	Keep away from sunlight
	Medical Device
	This Side Up
	Handle with care
	Keep Out of reach of children

Symbol	Meaning
	Maximum stacking
	Batch Code
	Do not use if package is damaged and consult Instructions for use
	Re-use
	Non-Sterile
	Catalogue Number
	Unique device identifier

MADHU INSTRUMENTS PVT. LTD.

Mfg. Facility: A 260, Okhla Industrial Area Phase-I, New Delhi-110020 India.
Regd. Add.: F-90/30, Okhla Industrial Area Phase-I, New Delhi-110020 India
Customer Care No.: +91-11-42701030; 42701031
Email: customercare@madhuinstruments.com
Website: www.madhuinstruments.com
M.L.: MFG/MD/2023/000843

Doc. No. MIPLM7/IFU/01, Rev. No.: 00, Rev. Date: 13.09.2024